

BOLUS INSTRUCTIONS

TO RECEIVE A FREE BALLING GUN CALL 1-800-466-8235




ANTIBODY DRIVEN IMMEDIATE IMMUNITY AGAINST BACTERIAL & VIRAL SCOURS

Step 1:

Insert Bolus into the First Defense® Balling Gun

- Place long skinny end into base of gun

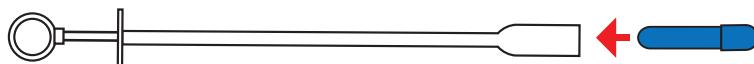


Step 3:

Insert Balling Gun

- Open the calf's mouth
- Insert balling gun
- Angle balling gun to the left until you feel it on your thigh
 - Your fingers on the balling gun handle should graze calf's teeth

Bolus needs to be in the feeding tube before ejecting from the balling gun. If the Bolus is ejected too soon (in the mouth/throat), it's like giving a child a marble - you risk choking.

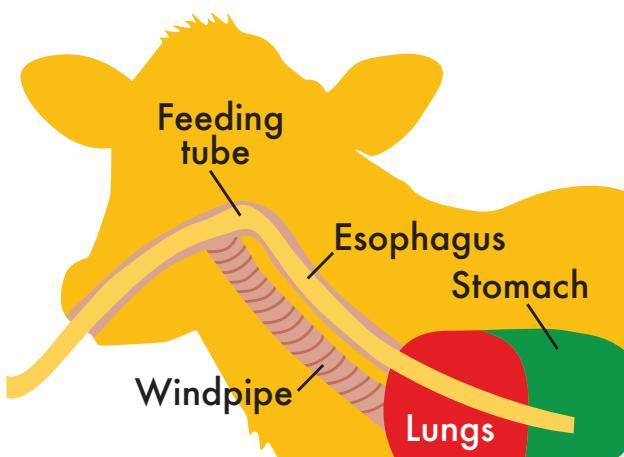

Step 2:

Secure calf


- Straddle the calf between your legs
- Hold calf against your thigh with head up

Step 4:

Release Bolus

- Release bolus by depressing balling gun plunger
- Keep calf's head up until he/she swallows
- Rubbing neck will induce swallowing reflex

Best sign a calf has swallowed the bolus: he/she will lick lips and nose

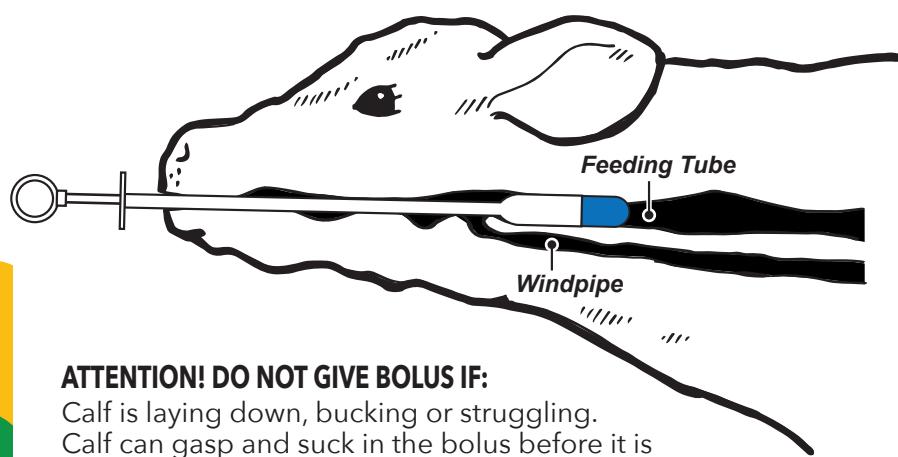

ATTENTION! DO NOT GIVE BOLUS IF:

Calf is laying down, bucking or struggling.
Calf can gasp and suck in the bolus before it is properly placed.

INSTRUCCIONES PARA ADMINISTRAR BOLOS

PARA OBTENER UNA PISTOLA PARA BOLOS EN FORMA GRATUITA, LLAME AL 1-800-466-8235




INMUNIDAD INMEDIATA POR ANTICUERPOS CONTRA LA DIARREA BACTERIANA Y VIRAL

Paso 1:

Introduzca el bolo en la pistola para bolos First Defense®

- Coloque el extremo largo y delgado en la base de la pistola

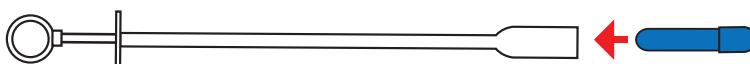


Paso 3:

Introduzca la pistola para bolos

- Abra la boca del ternero
- Introduzca la pistola para bolos
- Incline la pistola para bolos hacia la izquierda hasta sentirla en el muslo
 - Los dedos que están sobre el mango de la pistola para bolos deben rozar los dientes del ternero

El bolo debe estar colocado en el conducto alimentario del ternero antes de expulsarlo de la pistola para bolos. Si expulsa el bolo demasiado rápido (en la boca o la garganta), es igual que darle una canica a un niño: existe riesgo de asfixia.

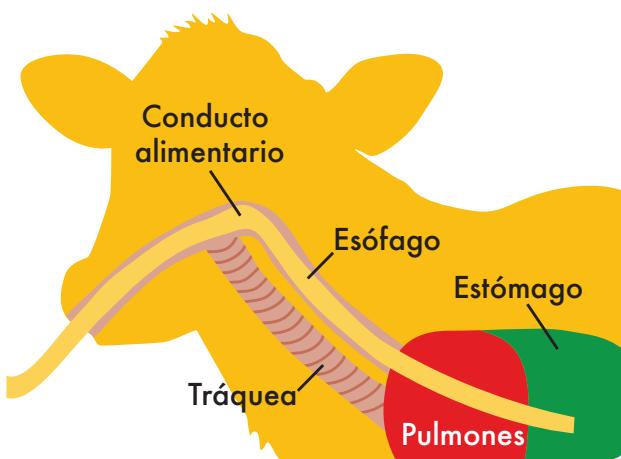

Paso 2:

Sujete al ternero

- Sostenga al ternero entre las piernas
- Sostenga al ternero contra el muslo con la cabeza hacia arriba

Paso 4:

Libere el bolo

- Libere el bolo apretando el émbolo de la pistola para bolos
 - Mantenga la cabeza del ternero hacia arriba hasta que lo trague
 - Frotarle el cuello provocará el reflejo de la deglución

El mejor indicio de que el ternero ha tragado el bolo: se lamerá los labios y la nariz

